Schlafmaske | Sleep Mask

09581 / 09582

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Hydas is responsible for complying with the guidelines of the Medical Devices Act; other product-related issues must be clarified with the importer and distribution partner DS Produkte GmbH.

*Anruf in das deutsche Festnetz zum Tarif deines Anbieters.
Calls to German landlines are subject to your provider's charges.

ID: Z 09581_09582 M DS V2.1 0821 mh

EN Instructions for Use

Dear Customer,
We are delighted that you have chosen our **mysleepmask**. Read through the following instructions for use carefully before using the sleep mask.

If you have any further questions, contact our customer service department via: www.dspro.de/kundenservice

SYMBOLS

Danger symbol: This symbol indicates possible dangers. Read the associated safety notices carefully and follow them.

Supplementary information

Read the instructions for use before using the product!

Reference number, international

09581

09582

30°C Delicate wash

Do not dry in a tumble dryer

Do not bleach

Do not iron

Do not treat with solvents

We recommend cleaning your mask before first use following the instructions on the washing label.

ITEMS SUPPLIED

- 1 x **mysleepmask**
- 1 x forehead gel pad (rectangle)
- 1 x neck gel pad (square)
- 2 x earplugs (pair)
- 1 x storage bag
- 1 x instructions for use

Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the product but contact our customer service department.

INTENDED USE

- mysleepmask** is intended to be used as a sleep mask and tension relief mask. Thanks to multiple fabric layers, a reinforced nose section and reinforced ear sections, it offers complete darkness and noise reduction. The gel pads which are also supplied (for cold and hot application) can be placed in the two pockets at the front and back.

- mysleepmask** helps you to find peace and sleep better when you have a migraine-induced headache. However, it is not a substitute for medically or therapeutically required treatment!

- The earplugs are suitable for reducing noise.
- For reasons of hygiene,
 - the earplugs should only be used by one person
 - the earplugs should be disposed of after being used once.
 - the **mysleepmask** should be cleaned regularly following the instructions on the washing label.

- Only use for the intended purpose. Any other use is deemed to be improper.
- The warranty does not cover faults caused by incorrect handling, dam-

age or attempted repairs. The same applies to normal wear and tear.

IMPORTANT NOTICES

- The gel pads are not suitable for people who are insensitive to the cold or heat.
- The earplugs of the product are not suitable for people who are sensitive to pressure on their ears.
- If you have concerns about your health or are regularly undergoing medical treatment, consult a doctor before using the product.

Risk of Injuries

- Keep children and animals away from plastic bags and films. There is a danger of suffocation!
- Be careful when removing the gel pads from the microwave or hot water, and use oven gloves or something similar if necessary. There is a danger of burns!
- If one of the gel pads is going to be used without the mask, protect the skin with a cloth. Do not use the gel pad in one spot for too long. There is a danger of burns from heat or cold.
- If you know you are intolerant to one of the components, the product and its accessories must not be used. There is the risk of allergic reactions!
- The product is **not** a toy! Keep it away from children and animals!
- If you experience any pain while wearing the mask, remove it and consult a doctor if necessary.

Risk of Damage to Material and Property

- Protect the product and all accessories from naked flames, direct sunlight, extreme temperatures and sharp or pointed objects.

- Do not overheat the gel pads, they could rupture. If they are damaged and gel is leaking out, wipe it up with a damp cloth as quickly as possible because in some circumstances it may leave stains. It is not toxic.


PREPARING THE GEL PAD

Cooling – Place it flat in the freezer compartment for around 1 hour or in the fridge for 1.5 – 2 hours.

Warming – Warm it either in hot water or the microwave:

- Microwave** – Wrap the gel pad in paper or a damp cloth, place it flat in the microwave and heat it up:
 - max. 30 seconds at 800 watts
 - max. 25 seconds at 1000 watts
 - max. 20 seconds at 1200 watts
- Hot water which is no longer boiling** – Place the gel pad carefully in water, heat it up for max. 5 – 7 minutes.

INSERTING THE GEL PAD

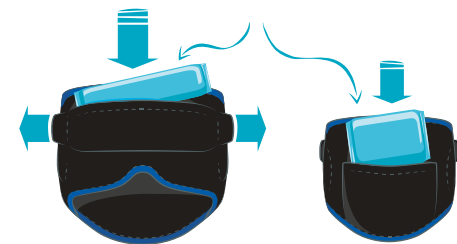Slide the **forehead gel pad** into the front of the mask. Make sure that it fits in the middle and is not compressed. To do this, reach into both sides of the pocket and pull the gel pad into the correct position.

Place the **neck gel pad** in the back of the pocket. Make sure that it fits straight and is not compressed.

INSERTING THE EARPLUGS

Take the earplugs between your fingers and use your body heat to taper them into shape so that they fit well in your ear.


Place one earplug in your right ear and one in your left ear. Wait a little while until they expand in your ear and then tighten their fit. Do not push them too far into your ear! The earplugs must fit securely, but must still feel comfortable. They must not press on.

PUTTING ON THE MASK

Do not pull it over your nose, but press the nose pads in against your nose to achieve the greatest possible darkness under the mask.

This is how it should look: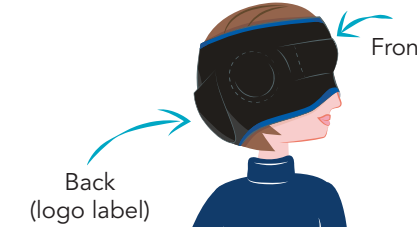

CARE INSTRUCTIONS

Please Note!

- Do not place the gel pads in the washing machine.
- Do not use any corrosive or abrasive cleaning agents to clean the product. These may damage the earplugs, the gel pads and the fabric of the mask.

- Clean your **mysleepmask** regularly following the instructions on the washing label.
- Clean the gel pads with a soft damp cloth and a little mild detergent if necessary.
- Dispose of the earplugs in an environmentally friendly way after use.
- Allow all components to dry completely before they are stored away.
- Keep your **mysleepmask** and all accessories in the storage bag and store them in a dry place which is protected from sunlight and is out of the reach of children and animals.

DISPOSAL

Dispose of the packaging in an environmentally friendly manner so that it can be recycled.

If the **mysleepmask** or individual components are no longer going to be used, each of the components should also be disposed of in an environmentally friendly way.

PRODUCT DATA

Mask

Size S/M
09581

Size L/XL
09582

Class 1 Medical Device.

Earplugs

Model designation: EC-1003A

Lifespan:

- unopened approx. 1 year
- when used: single use

Size range: Medium (7 – 12 mm)

Sound insulation values

Frequency (Hz)	Sound insulation M_i (dB)	Result		Requirement	
		Standard deviation S_i (dB)	APV $M_i - S_i$ (dB)		(dB)
63	27.2	6.1	21.1		
125	30.2	6.7	23.5		5
250	37.0	4.9	32.1		8
500	42.3	7.1	35.2		10
1000	41.7	7.1	34.6		12
2000	37.2	5.8	31.4		12
4000	47.8	6.3	41.5		12
8000	46.1	5.9	40.2		12
SNR value:		36 dB			
H value:		34 dB			
M value:		34 dB			
L value:		31 dB			

The earplugs comply with the European Standard EN 352-2.